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DF DentaFlux J. RIPOLL S.L.
Río Tiétar, 20
28119 Algeite • MADRID • SPAIN

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FAFC Vers 1 Rev. 04/18

and box.
Phases of danger
H300: Fatal if swallowed. H310: Fatal in contact with skin. H314: Fatal in contact with water. H314: Fatal in contact with water.
Safety advices
P260: Do not breathe dust / fume / gas / mist / vapours / spray. P264: Wash with water thoroughly after handling.
P301: Take off immediately all contaminated clothing. P301+P310: If swallowed: immediately call a poison

control center or a doctor. P405: Store under lock and key. P501: Delete the contents/container according to the current legislation.

almost all ceramic. DO NOT USE with aluminic ceramics.
Action
The gel of Acid Hydrofluoric "DENTAFLUX" is used for the preparation of the ceramic surfaces to have that produces, reinforces the adhesion between the cement (composite) and the surface of the ceramic.
Uses
Etching of ceramic surfaces.
Way of use
Apply on the ceramic surface without allow to act during 2 minutes.
Conservation
Eliminate the acid with abundant water. Dry.

It could be used so much in laboratory like in clinic.
It's suitable for ceramic crowns without metal, faces of porcelain, inlays/onlays of ceramic and brackets. Copable for

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