CLAVE CLEAN

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Clave Clean is formulated for cleaning and maintenance of autoclave chambers. It helps to remove debris and oxides and prolongs autoclave life. It dries quickly and is biodegradable.

Weekly cleaning is recommended.

HOW TO USE

Autoclaves without water tank: thoroughly clean the sterilization chamber with a sponge soaked in Rinse with water.

Perform a complete sterilization cycle.

Autoclaves with water tank (Class A).

1. Empty and clean the water tank, check that the injectors are not plugged.
2. Put 30-50ml of cleaning solution in the deposit.
3. Do a sterilization cycle in vacuum to clean up the system.
4. Empty the water tank with the remains of solution and refill with distilled water.
5. Clean with a sponge soaked in Clave Clean the sterilization chamber, then rinsing thoroughly with a cloth soaked in distilled water. If you like, it can be cycled with distilled water to rinse cleaner residue.

After this, the autoclave is clean and free of debris that can deteriorate it.

PRECAUTIONS

The product is intended for use in stainless steel. If your autoclave is of another type materials, must prove in each case.

Use in a ventilated room.

COMPOSITION

| | |
|-------------|-------------------|
| Surfactant | Isopropyl alcohol |
| Citric acid | Excipients |

WARNINGS

R10 Flammable.

R36 May cause eye irritation.

R67 Steam can produce drowsiness and dizziness.

S16 Keep away from sources of heat. No Smoking.

S24/25 Avoid contact with eyes and skin.

S26 In case of contact with eyes, rinse immediately with plenty of water and consult to a doctor

S7 Keep container tightly closed.