# SOFT PROTECT

*PROTECTOR GINGIVAL - GINGIVAL PROTECTOR - PROTECTEUR GINGIVAL - PROTETTORE GENGIVALE - PROTETOR DAS GENGIVAS*

( E )

## COMPOSICIÓN:

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Diuretane Dimetacrilate. Filler. Initiators.

## USE:

Fotocured resin fot two main applications:

- 1- Isolate the necks and protect the soft tissues in the light activated whitenings, where irritant substances are used.
- 2- Space creator for the whitening adaptations, so the whitening gel could be left inside them.

## INSTRUCTIONS:

Apply a cord over the necks of the teeth to whiten using the tips provided, and cure it with 20 seconds application of halogen light.

In the curing process so small heat is produced. It is recommended that after the first aplicacion of the whitening material to ask the patient for any sensitivity, those are points where the isolation can not be completely and more gingival protector is needed.

After finishing the whitening process remove the material.

To create space in the whitening applications, soft protect must be placed over the plaster model, on the vestibular face of the teeth to treat (usually from canin to canin). Place the material separated of the necks and of the oclusal area, and cure it.

This way a space will be created on the whitening adaptations and the gel could be kept inside.

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