# BARNIZ DE COPAL

( E )

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## USO ODONTOLÓGICO

( E N )

**INDICATIONS / PROPERTIES:** *Dentin Protect shield the pulp and dentin tissue:*

- Makes a barrier for the chemical irritants avoiding the penetration into the dentinal tubules.
- Temporary barrier to reduce the drying in and out of the surface obturations for example with glass ionomers.
- Protects the calcium hydroxide cements against the etching procedure.
- Helps to the sealant effect for example with amalgams.
- Temporary protector of the sensitive dentin and exposed cement.
- As of applicaion of topical fluoride method.

## INSTRUCTIONS FOR USE:

- Apply the Dentin Protect with a pellet, cotton roll or brush in several layers, leaving 30 seconds between the layers to hard.
- The solvent can be evaporated with an air flow. One only layer present often too high porosity, 2 or 3 produces an optimal seal.

- A soft bright on the dentin shows saturation of the varnish.

**PRECAUTIONS:** Clean with a solvent the rests of varnish on the enamel, to avoid alterations over the resins cure process. Caution to allergic reactions to copal resin. Its use is not fully recommended for glass ionomer seals.

**COMPOSITION:** Copal varnish and excipients.

**STORAGE:** Keep tightly closed, if not a quick evaporation of the solvent, harden the varnish or seal the cap of the bottle can be produced.

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The logo for DentaFlux, featuring the letters 'DF' in a dark square followed by the brand name 'DentaFlux' in a bold, sans-serif font.

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