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CONTENU:

Poudre: 40 g. • Liquide: 20 ml.

EN **ZOE-T** is a zinc oxide based reinforced cement for temporary restorations and bases.

WAY OF USE:

Spatulate over a glass surface adding parts of the powder to the liquid until all the powder is added. Place the cement into a dried and retentive cavity. If the pulp is very closed, a base liner of calcium hydroxide or better calcium hydroxiapatite as Fluoroseal should be used. Mix 3 spoons of powder, for 2 drops of liquid. Do not press the powder and remove the excess of powder with a spatula. The spoon is included in the kit. • If the mix is done on different proportions, the physical properties can vary.

MIX:

Mix over a glass surface by an spatula, adding to the liquid small parts of the powder until all is added. Spatulate until a uniform mass is obtained. Place the cement into a dried and retentive prepared cavity. If the pulp is very closed or exposed, a calcium hydroxide can be placed. If the occlusion should be controlled, use a matrix in the usual way. Working time has 2.5 minutes. Setting time has 5.5 minutes.

PRECAUTIONS:

The Zinc oxide cement can interfere in curing resins. In case of contact with eyes rinse and get medical attention. Keep the bottles tight closed and separated from sunlight. For the manipulation of the product, the use of gloves and glasses are recommended. After its application it is recommended that the patient does not exert pressure in an approximate time of 15 minutes.

COMPOSITION:

Powder: Zinc oxide, polymethyl metacrylate, zinc acetate. Liquid: Eugenol.

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