Instruction for Use

FLUOROSEAL

Light-Cured Cavity Base & Liner

General Information

Fluoroseal does not contains calcium hydroxide Ca(OH)_2 , but a calcium salt of apatite hydroxide $(\text{Ca}_{10}(\text{PO}_4)_6(\text{OH})_2)$. This compound has a biocompatibility long time tested in implantology procedures. The matrix of the formulation is Diurethane dimetacrilate, which provides the cured product with a hardness higher than the glass ionomers, as well with an excellent chemical union with composites. Another advantage of this matrix is the great thermal isolation. This thermal isolation is very important to avoid the heat produced in resins (composites) curing, reaches the vital pulp. Also a great resistance against chemical products is offered. It can be seen when it is exposed to etchers and no effect is observed. Fluoroseal is radiopaque for future RX texting. The dentin generation is produced when a strongly alkaline irritation, and Ca^{++} ions are available. The Apatite hydroxide calcium salt $(\text{Ca}_{10}(\text{PO}_4)_6(\text{OH})_2)$, is chemically closed with the well known calcium hydroxide (Ca(OH)_2) , also produces a strong alkaline medium, and frees Calcium ions. Anyway the secondary dentin formation has been observed clinically.

Composition

- Organic Matrix: Bis-EMA, UDMA, HEMA
- Inorganic Fillers: Calcium Hidroxiapatite, Micro / Nano silica
- Additives: Initiator, Catalyst, Stabilizer, Others

Indications

- Protect pulp of the tooth
- Lining under all filling materials

Precautionary information

- The product contains substances that may cause an allergic reaction by skin contact in certain individuals.
- To reduce the risk of all allergic response, minimize the exposure to this material, in particular avoid avoid exposure to uncured material. If prolonged contact occurs, flush with large amounts of water. If allergic reaction occurs, seek for medical attention as needed.

Application

- Isolate teeth threatened to avoid saliva and blood contamination.
- Prepare the cavity and thoroughly clean and dry the teeth again.
- If desired to apply a thick base of fluoroseal over the dentin, place first and cure a thin layer, of no more than 0.5mm. the curing process frees heat, and this thin layer protect from thicker ones.
- Cure the product with a suitable polymerization light.
- Sufficient exposure to the curing light prevents incomplete polymerization. For the recommendations regarding exposure time and light intensity see the following table.

Light Intensity (mW/cm ²)	Exposure Time (sec)
≥ 500	20
≥ 1000	10

- After light irradiation, continue with the usual procedure.

Caution

- Fluoroseal has been designed to be sold and used by licensed dental care professionals only. It has not been designed nor it is not suitable for use by non dental care professionals.
- Fluoroseal contains methacrylate monomer. Some patients can cause allergic reaction to monomer.
- Do not use in patients allergic to or hypersensitive to methacrylic monomers.
- If the product cause an allergic reaction or oversensitivity, discontinue the use immediately.
- Avoid contact with eyes, mucosal membrane, skin and clothing.
- If comes in contact with eyes, flush thoroughly with water and contact immediately an ophthalmologist.
- It is not allowed the use for different uses as indicated as well as by non professional dentist.
- Use a rubber dam to protect the throat.
- Do not use after the expiry date.
- If stored refrigerated, allow the product to reach room temperature prior to use.

Storage and use

The product has been designed for a storage at room temperature. If stored refrigerated, allow the product to reach room temperature prior to use. The shelf life at room temperature is 36 months. See package for expiry date. If the routine ambient temperature in higher than 27°C, it may reduce the shelf life, and in this conditions storage refrigerated is recommended. The eugenol can interfere with resin polymerizations.

Warranty

The manufacturer warrants this product will be free from defects in material and manufacture. The manufacturer MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PROPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period your exclusive remedy and the manufacturer's sole obligation shall be repair or replace the manufacturer's product.

Limitation of liability

Except where prohibited by law, the manufacturer will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental, or consequential regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Customer information

No person authorized to provide any information which derives from the information provided in this sheet.

Disposal

It should be handled according to local laws and regulations.

Conformity with Communauté Européenne			
	Temperature limit		Batch code
	Caution		Date of Manufacture
	Keep dry		Use-by date
	Keep away from sunlight		Consult Instructions for Use
	Manufacturer		Medical Device
	Authorized representative in the European community		
	Federal law prohibits dispensing without a prescription		

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